HEINE QUALITY
MADE IN GERMANY

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General Conditions of Warranty

Instead of the statutory warranty time period of 2 years, HEINE will grant a guarantee of 5 years from the date of the consignment of the goods ex works, concerning its equipment (excluding disposables, e.g. bulbs, single-use articles, and rechargeable batteries). Date of consignment means that HEINE hands the goods over to the transport carrier, freight forwarder or any other person designated by the Customer for the transport of the goods without loading the collecting vehicle.

The guarantee covers irreproachable workmanship, on condition of the proper use of the equipment and the observation of the operating instructions. During the warranty- and guarantee time period, errors and deficiencies arising on the equipment will be rectified free of charge, in so far as such are evidenced by defective materials, processing and/or constructional errors. Should buyer complain of a material deficiency during the warranty time period, then the onus of proof is always to be on the orderer, that the product was defective already upon receipt of the goods. The statutory warranty and the guarantee do not apply to loss or damage caused by wear and tear, negligent use, the non-employment of original HEINE components and/or spares (in particular bulbs, as these have been especially developed for HEINE instruments in accordance with the following criterions: colour temperature, useful service life, safety, optical quality and performance. The statutory warranty and the guarantee do not apply to interventions by persons not authorised by HEINE or when the operating instructions are not observed by the customer. Any modification of a HEINE product with parts or additional parts which do not conform to the original HEINE specification will invalidate the warranty for the correct function of the product and further invalidate any warranty claims which result from such a change or modification. Further claims, in particular claims for replacement of loss or damage, which are experienced otherwise than directly on the HEINE product itself, are hereby excluded.

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DEUTSCH

HEINE Finoff transilluminator

 Read these instructions carefully before using the HEINE Finoff transilluminator and keep them in a safe place for future reference.

 In accordance with local regulations this product should be disposed of as an electronic device separately.

Safety information

Range of applications: The HEINE Finoff transilluminator is used for general transillumination of tissue and, more specifically, for scleral transillumination. It can be combined with a blue filter for illuminating the eye.

Safety in use: Only power sources which are approved for medical use should be used. Apply the transilluminator to the eye only when the protective cap is fitted.

Please note: The performance of this instrument can only be guaranteed if genuine HEINE rechargeable batteries or alkali manganese batteries are used.

Instructions for use

We recommend that the examination be carried out in a darkened room. Ensure that the surface of the eye is anaesthetized before scleral transillumination.

- Connect the HEINE Finoff transilluminator to the handle and switch on.
- For scleral transillumination, rest the distal end with its protective cap flat on the bulbus without exerting pressure. Please note that intense light can be uncomfortable for the patient and may make it difficult to recognize areas of thin tumor growth.
- For transillumination of skin tissue use the instrument without the protective cap.
- To illuminate the eye, fit the blue filter and move the instrument as close as necessary to the eye without touching it.

Changing the bulb

Please note: The performance of this instrument can only be guaranteed if genuine HEINE bulbs are used.

First remove the instrument from the handle. Grip the used bulb by the collar (1) or the thread (2) and pull it out of the instrument.

Care and maintenance

After removing the protective cap or blue filter, you can clean the instrument with a cloth soaked in a mild cleansing agent or with a disinfectant solution. Gas sterilization is also possible. Soaking in solutions and autoclaving are not possible. The protective cap and blue filter can be cleaned and sterilized by any conventional means.

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SIR Oftalmica Srl